



Pg Supply Instructions for Use: Orthodontic Wires

Intended Use - General

Orthodontic wires, bands, brackets and tubes are meant to be used together to create a system that applies force to a patient's teeth in order to move them into proper alignment. Each has a specific use and each has multiple variations to accommodate the variety found from patient to patient in both size and application. Pg Supply offers orthodontic wires and springs.

Intended Use - Orthodontic Wires



Orthodontic wires are used to deliver force on the teeth to move them from malposition to an intended correct position. Other uses of orthodontic wires are stabilizing teeth which are in good position. Loops, and springs can also be made with the orthodontic wires to deliver lighter and longer acting force.

Use of Product

Orthodontists determine the appropriate devices and materials for each stage of the patient's treatment.

Wires and orthodontic brackets are intended for single use only. Re-use of items will not generate the forces needed to move teeth according to the treatment plan that the doctor has established.

Manufacture

All products are manufactured out of stainless steel, titanium or nitanium wire or strip. They are formed, stamped and may go through a series of secondary operations (heat treat, weld, etc.) to achieve the necessary configurations.

Product Information

	Identifies the company listed as the manufacturer.
	Identifies the authorized representative in the European Community
	The catalog / part number.
	The batch code.
	Warning Orthodontic wire devices are intended for Single Use Only. Reuse of an orthodontic device may impair product performance.
	Warning This product contains nickel and should not be used by or for individuals with an allergic sensitivity to nickel.
	Date of Manufacture.
	Use by Date.
	Instructions For Use.

Precaution Orthodontic Wire Devices have not been evaluated for compatibility in an MRI environment

Storage and Handling: Orthodontic devices are sold non-sterile. They may be single or bulk packed. No special storage conditions are required.



Pg Supply Instructions for Use: Orthodontic Wires

Incident Reporting: Any serious incident that has occurred in relation to these devices should be reported to Pg Supply (contact info. below) and the competent authority of the Member State in which the user and/or patient is established.

Additional Information: Refer to the Pg Supply website and or sales literature. www.pgsupply.com

 Pg Supply 26 Freedom Drive Collinsville, Ct 06019 USA (860) 673-4155	<table border="1" data-bbox="940 432 1081 485"><tr><td data-bbox="940 432 1003 485">EC</td><td data-bbox="1003 432 1081 485">REP</td></tr></table> EC Certification Service GmbH Sandgasse 39a, A- 9300 St. Veit/Glan Austria 43 42 126094	EC	REP
EC	REP		